

# **SPINAL REMODELING BRACE-THE ANTRO VERSION**

## **Traction Sling and Antro Roll Placement**

If you have purchased the Antro Version of our Spinal Remodeling Brace and have a patient that presents with anterior thoracic translation posture with increased pelvic/sacral anterior tilt, the traction sling should be tightened down across the posterior hips at the gluteal fold.



The Antro Roll should be positioned across the superior ASIS's and pushed farther posterior after the patient has walked and warmed up in the brace for 1-2 minutes.

